

---

## TD Snap

### Editing Buttons

---

### Edit Mode


To make changes to a button, the software must be in **Edit Mode**.



Select this **Edit** button, located on the top toolbar to enter Edit Mode.

### Button Editing Tools

The tools on the right side of the app window help you to edit the currently selected button(s).

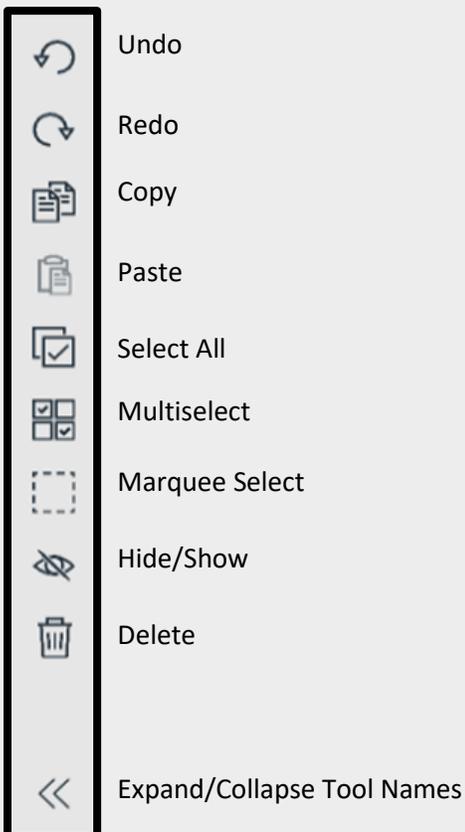


## TD Snap

### Editing Buttons

#### Editing a Button

1. Enter **Edit Mode** by pressing the **Edit** button on the top of the toolbar.
2. Select a Blank button or the Button you want to edit.
3. When you are finished editing, click **Done** at the top right corner.

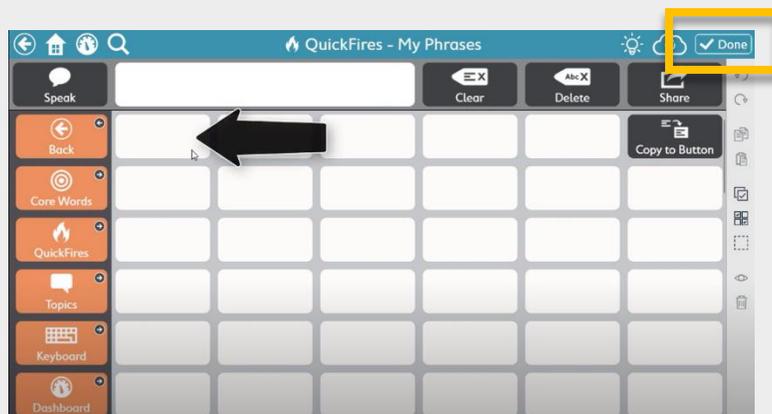


#### Components of a Button



1. **Button Label**
2. **Message**
3. **Symbol**
4. **Action**
5. **Style**

## TD Snap

### Editing Buttons

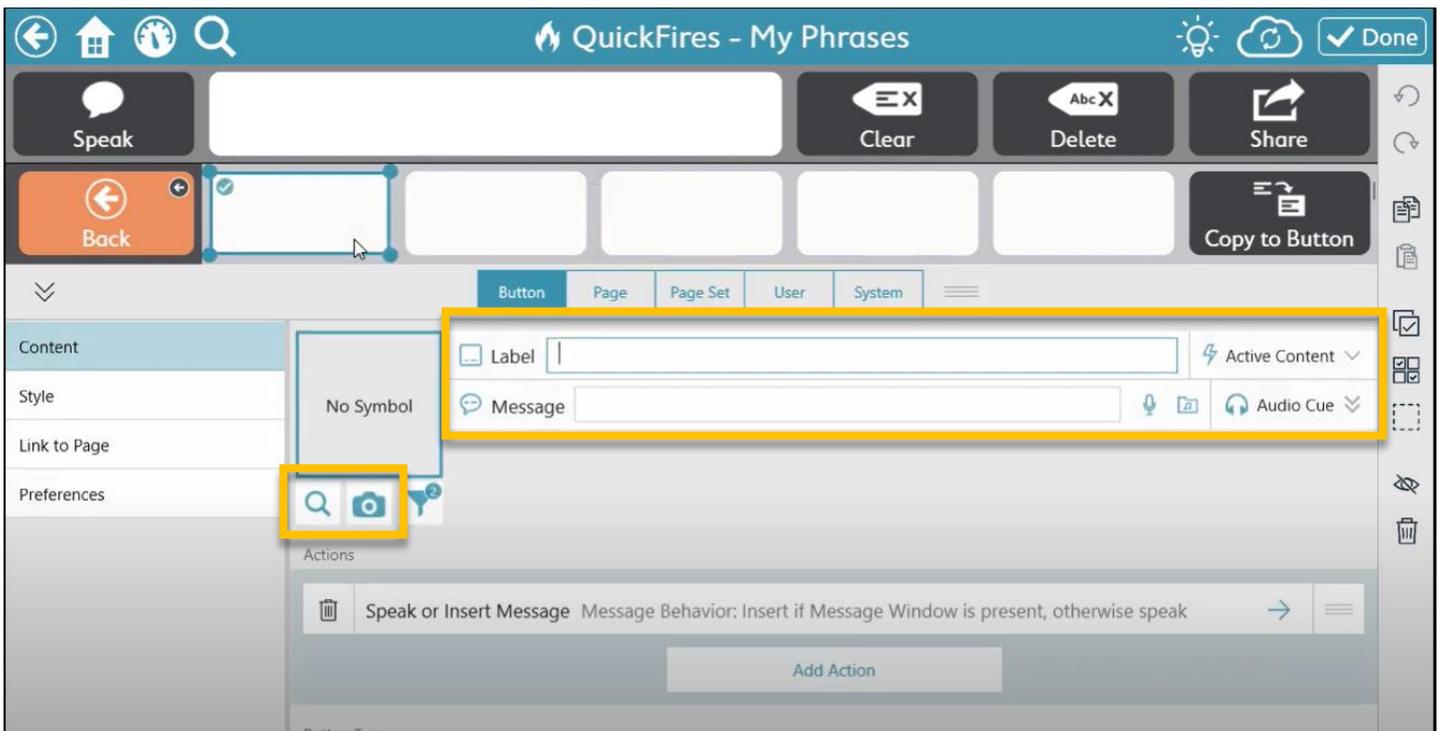

#### 1. Edit the Button Label

The Button Label is the text that's visible on the Button.

#### 2. Edit the Message

The Message acts at the user's voice. This is what's heard when using speech output. This automatically matches the Button Label, but you can edit it by selecting the text in the Message box and typing.

---

## TD Snap


### Editing Buttons

---

### 3. Symbol



To search for **Symbols** - Select the Symbol or Magnifying glass to search for a symbol.

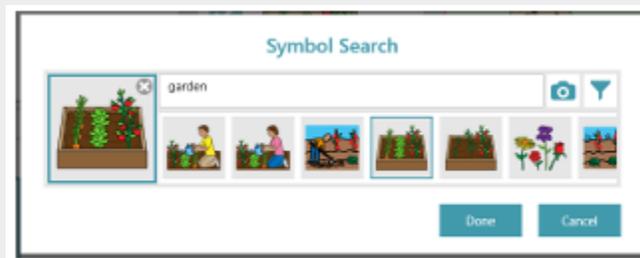

To search for **Real Pictures** – select the Camera icon to take a picture **From Camera** or **From Photo Library** from pictures stored on the device.


## TD Snap

### Editing Buttons

#### 4. Action

Actions are what the Button will do when it is selected. The default action for the Button is to **Speak or Insert Message** in the window button.

The Arrow Button means there are options. Select this button to see the options and to make a change.

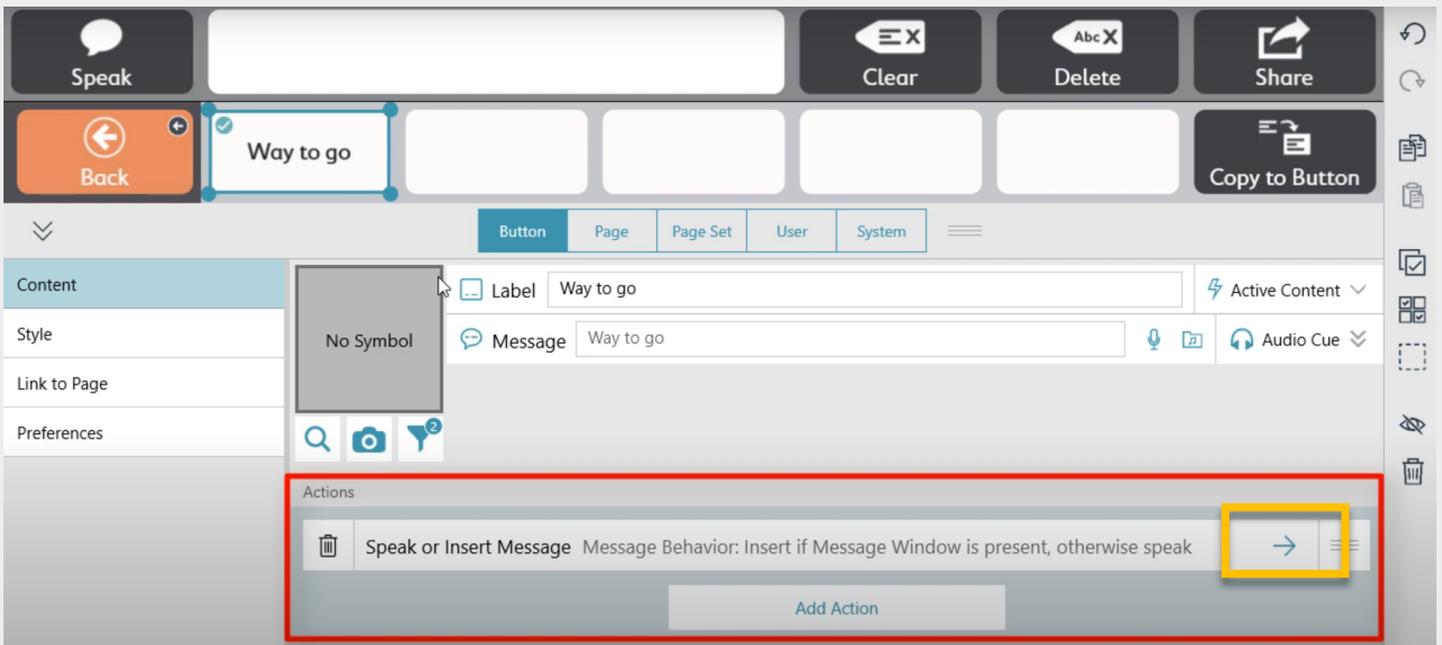

## TD Snap

### Editing Buttons

#### 5. Style

You can change the **Button Background, Border, Font, and Layout**. Select the options you want to change.

